
NEW SKEMA 8

REWRITES IH.

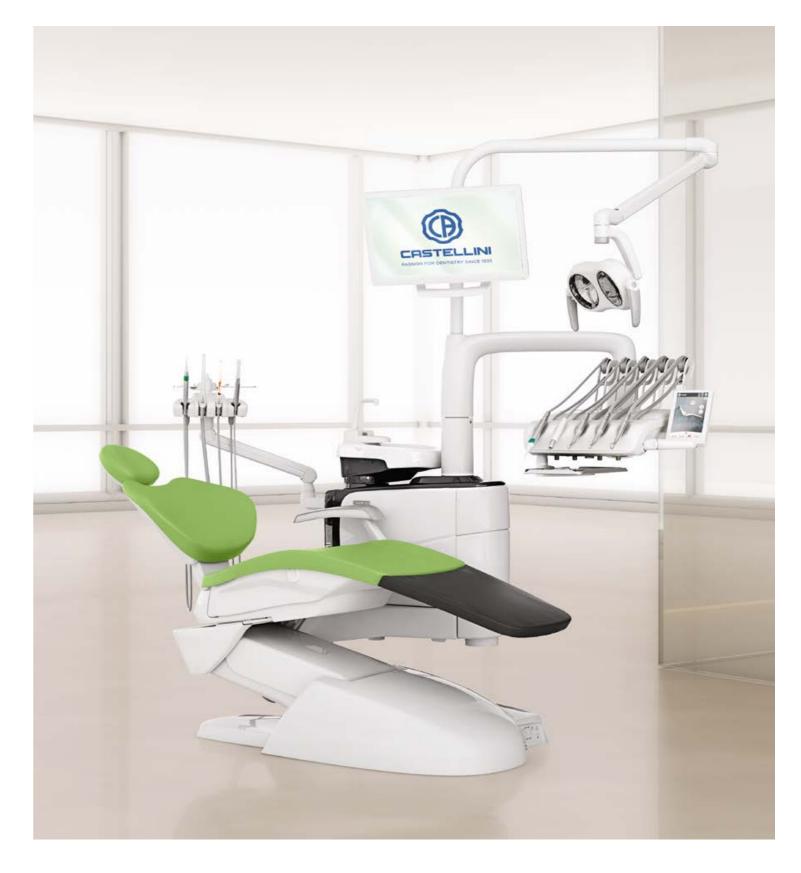

NEW SKEMA 8

REWRITES THE FUTURE







THE HIGHEST EXPECTATIONS EVER

True to Castellini's renowned quality, improved in every aspect. Skema 8 embodies the revolution that rewrites the future of dentistry. Designed to ensure optimum performance thanks to its cutting-edge technology and complete restyling, Skema 8 is reborn to meet the highest expectations ever set.

The vision of the future has become a reality.



SUPERIOR ERGONOMICS IN ACTION




SLIDING INNOVATION

Skema 8 provides extraordinary functionality as a result of leading-edge research. The innovative sliding system, which allows the seat to slide forward while lowering the backrest, maintains the operating zone unchanged thus the dentist has no need to move the accessories when performing the treatment. Besides increasing the comfort for the patient thanks to a reduced compression on the back, the Sliding function makes Skema 8 the perfect patient chair for small dental practices. Thanks to a seat displacement of 18cm, the doctor gains as much in terms of operating space in the 12 o'clock position, while the Soft Motion technology makes movements smoother and noiseless.

The new adjustable footrest allows for an easy and accurate disinfection.



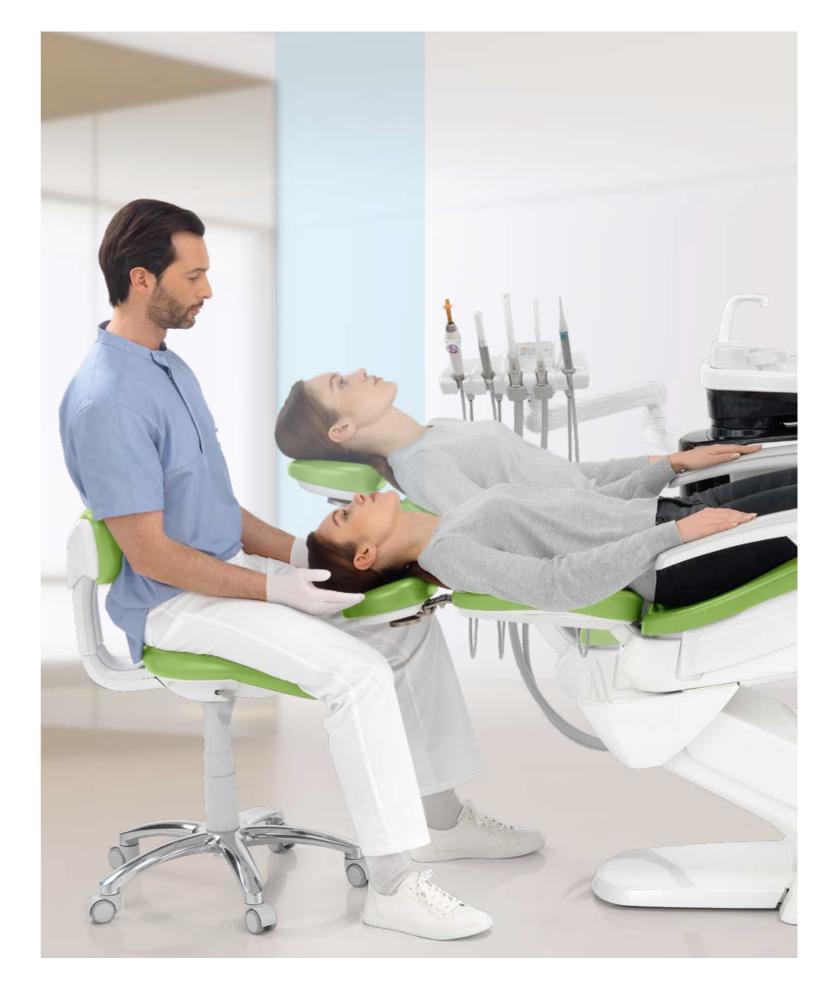

UNIQUE PERSONAL **EXPERIENCE**



The Original and Nordic backrests provide two excellent ergonomic solutions depending on the doctor's working method. The Original variant facilitates access to the operating zone both for frontal and indirect views, while the Nordic version is characterised by a shape that ensures optimum support for the patient's arms. Furthermore, the narrow upper backrest allows the dentist to gain closer proximity to the patient.

The surgical padding reduces the pressure points on the patient's body, favouring blood microcirculation and ensuring greater comfort.



Being the perfect synthesis of aesthetics and functionality, the backrests combine a brand-new comfort experience with the best technological solutions. Available in two versions - Original and Nordic - the backrests feature a highly resistant structure with soft padding and an extremely robust and easy-to-clean upholstery. This level of customisation makes it possible to adapt the dental unit to the doctor's



specific needs and habits.

PNEUMATIC HEADREST

Both backrests feature the exclusive pneumatic release orbital headrest which pivots on three axes and allows for a quick and precise positioning. The button on the lower side of the headrest activates the movement.



LATERAL SLIDE FOOT CONTROL

The pedal can be used to activate the Chip Air and Chip Water functions of the instrument in use, reverse the micromotor sense of rotation (when deployed), move the In surgical mode, both pedals feature the patient chair and switch on the lamp.



PRESSURE-OPERATED FOOT CONTROL

This pedal provides the same functions of the lateral slide foot control to those who prefer the pressure-operated version. controls to activate the peristaltic pump.



POWER PEDAL FOOT CONTROL

The third pedal includes all the functions of the previous models and is characterised by two additional elements to recall two more stored working positions.

THE BEGINNING OF A NEW DESIGN




LIGHT&EASY MODULE

The dentist's module with over-the-patient delivery is light and easy to move.

- · The vertical positioning can be activated and deactivated by lightly touching the capacitive sensor.
- · Increased tubing extension capacity.
- · Large and easy-to-access transthoracic tray holder.
- · Self-balanced instruments to minimise dentist wrist fatigue.
- · Optional end-of-stroke lock system for enhanced freedom of movement.



With a radically innovative concept, Skema 8 stands out for its captivating design with shapes that make it extremely compact. Everything has been designed to maximise the functionality of each element, and its clean and essential lines, combined with the best technologies available on the market, represent a great revolution.

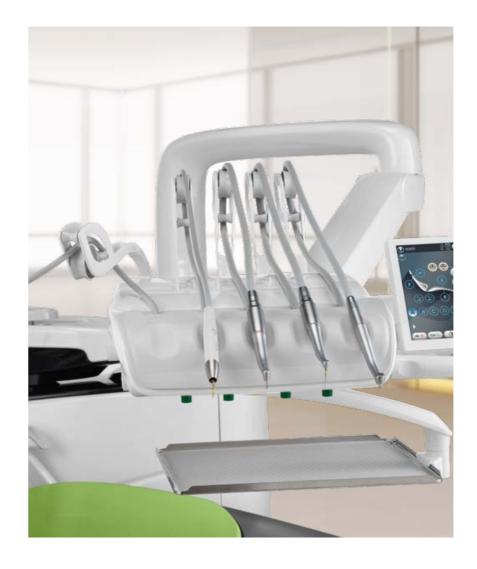

LEVERS WITH SIDEFLEX TECHNOLOGY

The instrument levers have small overall dimensions with a wide extension and sensitive balancing. Optimised with the exclusive SideFlex technology, the levers follow the tube's lateral movements thanks to the presence of a tilting joint. The levers also ensure accurate delivery without exercising traction on the operator's wrist and can be completely disassembled.



COMPACT TRAY HOLDER

The new compact tray holder features an essential design and small footprint, thereby instruments are light and easy on recall. It ensuring the operator maximum freedom of is possible to adjust balancing and traction movement.



BALANCED INSTRUMENTS

Thanks to the lever's articulated joint, the during position recalls.

THE AESTHETICS OF PERFORMANCE

ASSISTANT'S MODULE

The assistant's module, fitted on a vertically-adjustable double arm, adapts to any working method and is available with 4 or 5 housings.

CUP FILLING SENSOR AND MOTOR-DRIVEN CUSPIDOR BOWL

Cup filling is now completely automated, thereby giving more autonomy to the patient. Together with the new motor-driven cuspidor bowl, it makes rinsing extremely fast.



Skema 8 reveals a close attention to the design and manufacturing of every technical and design element. Starting with the innovative carbon look of the cuspidor bowl, every detail of the dental unit has been developed to create a comfortable and pleasant setting, rich in technology and functionality: a modern practice where dentists can fully express their expertise.



AN ASTONISHING REVOLUTION

Skema 8 control panel opens a new technological era. It features a 7" capacitive multitouch display that makes it easier to read multimedia content and data from every angle. Thanks to a more user-friendly interface, the innovative control panel ensures excellent performance and guides the operator with instructions on the wide range of clinical functions available.



UPGRADED TECHNOLOGY

- · 7" capacitive multi-touch display
- Water and shock resistant
- · Compatible with multimedia content



CUSTOMISATION SYSTEM

- Customisation of different operator profiles on the same dental unit
- Possibility to save operator profile settings on a USB stick
- Complete customisation of user interface

The innovative control panel makes it possible to control the operation of instruments, patient chair and integrated systems in a simple and functional way, as well as to assist dentists of all clinical specialities, from conservative dentistry to oral surgery. Ideal for both small practices that have a greater need to optimise processes, and for large multifunction clinics thanks to the high level of interface customisation. The new control panel is an advanced tool that contributes to increasing and improving the dental practice clinical treatments.



FULLY INTEGRATED IMPLANTOLOGY

Skema 8 facilitates all implant surgery and operating protocols with a fully integrated implantology system, thereby allowing the doctor to use an easier implant technique and to focus exclusively on the task at hand and on the patient. The implantology configuration consists of a powerful Implantor LED induction micromotor, a peristaltic pump integrated into the dentist's module and a cutting-edge control panel that makes it possible to quickly customise and store the speed, torque and delivery rate parameters of the pump.





GOLDSPEED EVO R20-L CONTRA-ANGLE

Specific for surgery. Autoclavable and thermodisinfectable, it can be disassembled for thorough cleaning. With internal cooling, external spray and LED with integrated generator.



IMPLANTOR LED MICROMOTOR

Noiseless and ultra-light, the Implantor LED micromotor is vibration-free and characterised by extremely high torque. With suitable contra-angles, it can deliver up to 70Ncm.



PERISTALTIC PUMP

Integrated into the doctor's module, it can be easily controlled from the control panel. The device irrigates the operating field with a sterile solution for optimum asepsis.

ACCURATE ENDODONTICS AT ALL TIMES

With Skema 8 all endodontic procedures and related treatments are supported thereby providing a significant help to the doctor. Thanks to a safe and precise control of the endodontic bur, to the integrated bur database and to the perfect control of specialised functions, Skema 8 becomes a must-have for endodontics. Furthermore, the dental unit provides the dentist with the most sophisticated control parameters, such as Autostop, Autoreverse and Autoforward.







RECIPROCATING MODE

Together with the EVO E4 contra-angle, the control mode with alternating rotation allows for the use of RECIPROC®, RECIPROC optimum parameters thanks to the inbuilt BLUE® and WAVEONE GOLD® burs.

DATABASE

For all traditional endodontic procedures, the control panel is able to automatically set database of the most common endodontic





LAEC - INTEGRATED APEX LOCATOR

The device helps the doctor locate the root apex through an audible and visual signal. Combined with the EVO E4 contra-angle, it also integrates the APEX-STOP function which automatically interrupts bur rotation upon reaching a set safety threshold.

EVO E4 CONTRA-ANGLE

Suitable for any type of endodontic treatment, the device is designed to be compatible with Castellini reciprocating mode and with the APEX-STOP function of Castellini apex locator.

HIGH SPECIALISED **PERFORMANCE**



PIEZOLIGHT 6

Available with fibre optic for LED lighting, the versatile and multifunctional ultrasound handpieces are suitable for prophylaxis, endodontics and periodontal procedures.

- · Perfectly linear and controlled ultrasonic vibrations free from parasitic lateral vibrations - with constantly self-regulated frequency according to load and tip conditions
- Extensive power adjustment range
- · Self-regulating power based on the workload to prevent heating of handpiece and treated tissues
- · Coolant delivery adjustable from zero to max flow rate
- · Completely autoclavable



Manufactured to the highest technical standards and built on the valuable experience gained by Castellini over more than 80 years of history, all the instruments, handpieces and accessories are designed to meet every dentistry need. Accurate, reliable, light and easy to handle, these devices are able to improve any conservative dentistry, implant surgery and endodontic procedure.



IMPLANTOR LED MICROMOTOR

Latest-generation brushless micromotor, with its low-noise operation and minimal weight, Implantor LED is vibration-free and characterized by extremely high torque and outstanding reliability. Top performance for all specialist applications.

Completely autoclavable.

Micromotor features:

- 100 up to 40,000 rpm with 1:1 ratio
- · electronic torque control up to 5.3 Ncm
- · possibility of pre-setting maximum speed



The curing light can be used as a direct version or 120° angled version. With a high-power LED up to a maximum of 2,200mW/cm² and a 430 to 490nm emission band, the T-LED lamp offers 6 preset programmes and an optical guide that can be sterilised.



Featuring a streamlined and easy-to-handle design, the syringe provides six functions. The handle and tip are autoclavable.



SILENT POWER TURBINES AND GOLDSPEED CONTRA-ANGLES

Castellini offers a complete set of turbines and contra-angles suitable for any field of

CONSTANTLY PERFECT LIGHTING



EXCEPTIONAL BRIGHTNESS

The new Venus LED MCT lamp rotates on three axes to ensure a perfect lighting of the operating field in any kind of treatment and position. Featuring exceptional brightness, it makes it possible to set specific values depending on the clinical needs.

The No Touch sensor switches on/off and adjusts the luminous intensity up to 50,000 Lux. The hermetically sealed front protective screen is easy to clean, and the handles can be removed and sterilised in autoclave.



A clear view, and hence the quality of light, are essential for the dentist who has to be able to always perform in perfect artificial lighting conditions. Skema 8 proposes the new Venus LED MCT, a high-performance lamp. This operating light works with a long-lasting and energy-efficient LED light source, and is characterised by a three-axis rotation as well as by the possibility to make several light adjustments.



TEMPERATURE ADJUSTMENT

The innovative system makes it possible to illuminate with the right light every clinical discipline, with 3 different temperature values: 4,300K, 5,000K and 5,600K.

CRI VALUE

Absolute colour fidelity thanks to a colour rendering that on a 0 to 100 measuring scale stands above 95.

COMPOSITE MODE

The quick and easy activation of the Composite mode prevents the precuring of composite materials.



WARM, NEUTRAL AND COOL COLOUR

All the temperature variations, from warm to neutral or cool and vice versa, can be easily activated and always ensure a perfect view of the operating field according to the treatment being performed.



CURING MODE

This feature changes the light wavelength to prevent the pre-curing of composite materials while ensuring optimum lighting of the operating field.

SAFE AND CERTIFIED **HYGIENE**



M.W.B.

Reliable and efficient, the M.W.B. system, used in conjunction with Peroxy Ag+, acts against all water-borne contaminants, including Legionella. Tests carried out on a dental unit over a 3-month observation period showed that a regular use of M.W.B. and daily performance of an intensive disinfection cycle with the Autosteril system, ensure outstanding water hygiene quality, with bacterial loads kept at zero.





The constant synergy with the most prestigious university centres in the field of clinical research its one of Castellini's cornerstones since its inception. Providing patients with a perfectly safe clinical environment is one of the most delicate and important tasks for a dentist. That is why efficient and easy-to-apply hygienic devices facilitate the activities of a dental practice and optimise its work-flow.



AUTOSTERIL

The new Skema 8 dental unit is equipped with a high-end system for the disinfection of all the spray water circuits. Provided with an inbuilt receiving area, it facilitates instruments positioning and cycle performance.

Used together with Peroxy Ag+, the Autosteril system ensures a mycobactericide, bactericide and anti-Legionella action, certified by the Sapienza University of Rome. Autosteril helps hygienisation at the end of a day's session and allows for short cycles to be carried out in the following steps:

- · circuit draining with compressed
- addition of 3% H₂O₂ to the spray water circuits;
- · 'hold time' allowed for liquid to work to intensive disinfection results:
- · liquid draining with compressed air;
- · water flushing (either from mains or distilled).



TIME FLUSHING

To drain stagnant fluid inside the piping out a complete water exchange with a timed flow of the supply fluid.



S.S.S. BOTTLE

The stand-alone supply system is equipped after use, the Time Flushing feature carries with a 1.8 litre capacity pressurised tank, easily accessible from outside to facilitate filling or replacement.

FULL AND EFFECTIVE **PROTECTION**

To maintain outstanding levels of hygiene on both the dental unit and in the workplace, Castellini has developed an advanced passive protection concept. Thanks to the careful design of every component of Skema 8, patients and medical staff are constantly protected against any risk of contamination.





The flushing and hygienisation of the suction system with water and a special liquid take approximately 2 minutes: the ideal timing for a cleaning procedure between patients.



DETACHABLE INSTRUMENT TUBING

Removable and easily disinfectable, the tubes make cleaning and replacement operations easier and faster.



REMOVABLE DENTIST'S MODULE HANDLE

It can be easily removed for maximum hygiene.



REMOVABLE LAMP HANDLES

They can be easily removed and cleaned in Disposable film for the protection of autoclave.



GLASS DISPLAY FILM

assistant's keypad and Full Touch control



CUSPIDOR BOWL

The cup-and-bowl-fill unit can be easily removed and sterilised in autoclave. The cuspidor bowl is also removable and autoclavable.



FILTERS + UNIT COVER

Smartly protected by a cover, the filters can be easily removed to facilitate draining and cleaning operations.

EXCELLENT MULTIMEDIA COMMUNICATION

Castellini multimedia system is the result of a reliable, efficient and cutting-edge technology and consists of an inbuilt camera with HD sensor and 90° viewing angle, as well as a 22" monitor certified by the EEC Medical Devices Directive. It provides an excellent multimedia communication that allows to simplify doctor-patient relations and makes treatment results more effective and understandable.




INTEGRATED HD CAMERA

The integrated camera features a streamlined design for a better use inside the oral cavity, and an ergonomic handle. HD-quality images ensure perfect rendering and maximum investigation accuracy.

- Optical system with 7 glass lenses and 16:9 format sensor
- · Partially retroflexed 90° viewing angle
- · Depth of field from 5 to 70mm
- · Freeze-frame activation through the touch-sensitive key



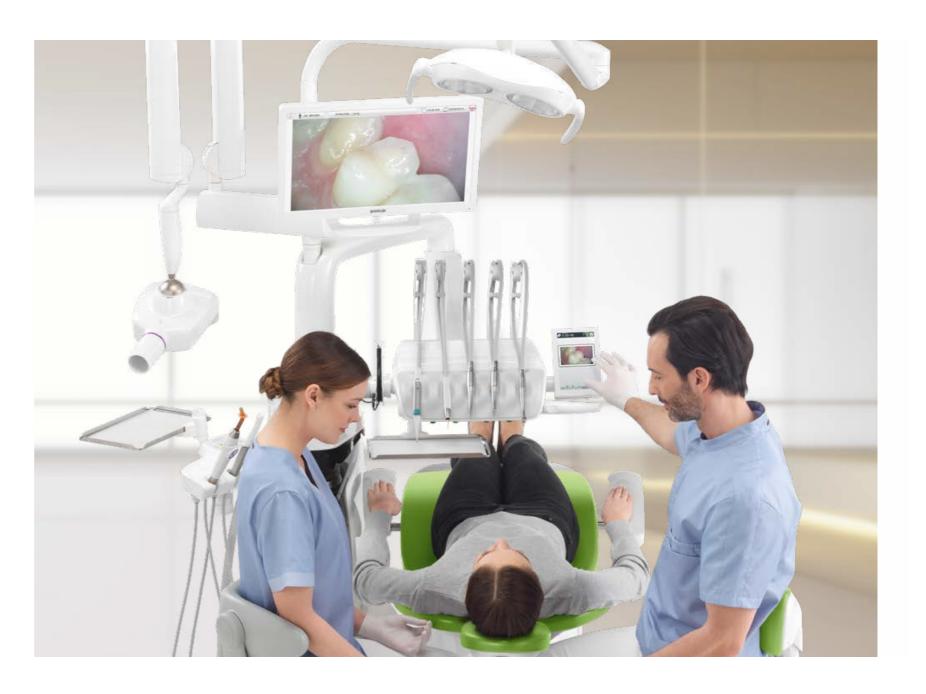

22" LED MONITOR

The 22" LED monitor with 16:9 format, Medical Device 93/42 certified, ensures a perfect view of the images, as well as an easy and comfortable interaction thanks to its ergonomic design and to the special IPS panel that provides a wide viewing angle. The images are managed by the new Full Touch Multimedia control panel that allows for the adjustment of the image contrast levels, colours, rotation and zooming. The monitor can be integrated into the dental unit either with a single arm or a double-jointed arm for a greater involvement of the patient and a more comfortable use.

NEW CASTELLINI DIAGNOSTICS




RXDC, HYPERSPHERE TECHNOLOGY


Thanks to its ball joint, the X-ray unit is able to fully rotate and easily reach any position. The unit produces high-quality imaging and allows the doctor to make more accurate diagnoses.


The integrated Zen-X sensor, available in two sizes, is housed in the dentist's module, thereby making the acquisition of intraoral X-ray images faster. In a matter of seconds, the reprocessed image is displayed on both monitor and control panel.



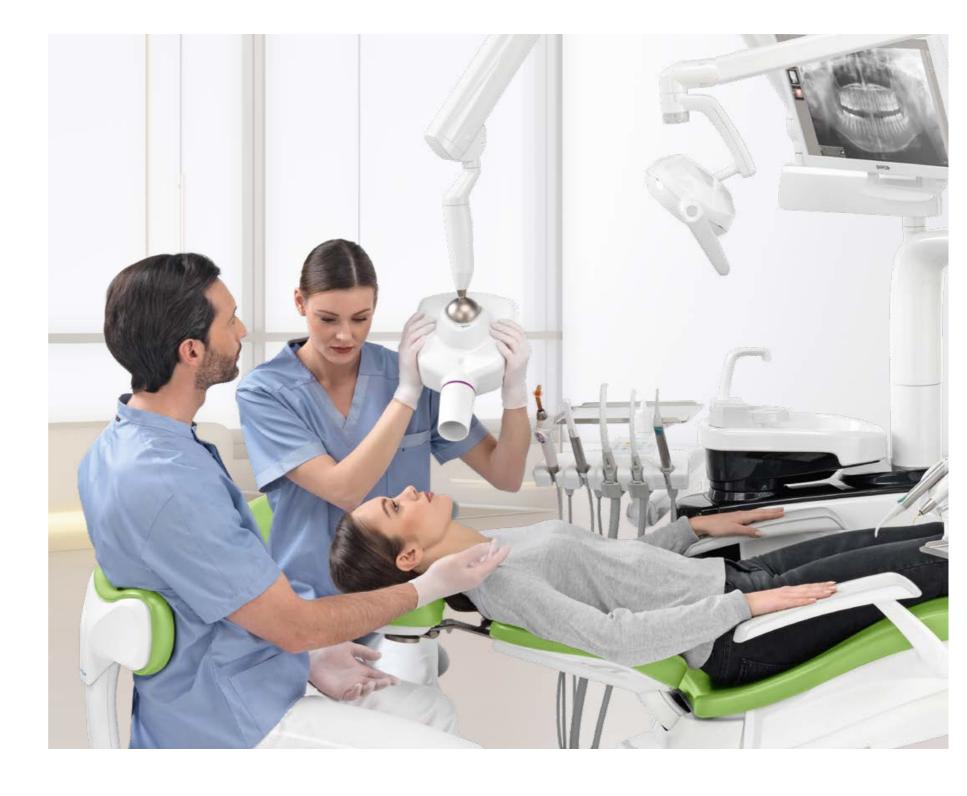

TECHNICAL DATA

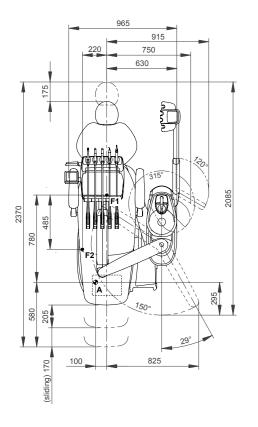

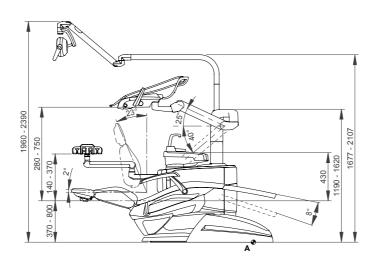

The measures in the drawings are in millimetres.



COLOUR RANGE

- 102 Atlantic blue
- 106 Mediterranean blue 113 Pacific blue

- 132 Blueberry violet115 Scottish salmon
- 123 Polynesian green
- 121 Anthracite grey
- 130 Graphite black

SURGICAL PADDING 198 Atlantic blue

- 196 Mediterranean blue
- 192 Blueberry violet
- 195 Scottish salmon
- 193 Polynesian green
- 199 Anthracite grey

SKEMA 8	OVER THE PATIENT DELIVERY
HYGIENE SYSTEMS	
Separate supply system	•
Autosteril A (Automatic) for disinfection of the water circuit	0
Multi Water Bio controller - MWB. (EN 1717-compliant water infeed unit with continuous disinfection system)	0
Time Flushing (quick wash cycle of the water circuit)	0
Suction Hoses Disinfection (Wash and disinfection of suction system)	0
DENTIST'S MODULE	
Instrument lever system with SideFlex technology	•
Turbine kit complete with Silent Power Evo 4L fibre optics, quick coupling handpiece	•
Turbine kit complete with Silent Power Evo Miniature fibre optics, quick coupling handpiece	0
Induction Micromotor Implantor LED fibre optics kit (100-40,000rpm - 5.3Ncm)	•
Peristaltic pump kit with saline solution irrigation set	0
Piezosteril 6 scaler kit complete with 4 tips	0
Piezolight 6 scaler kit with fibre optics with 4 tips	0
LAEC (Integrated Apex Locator)	0
	_
Module for Implantor LED motor Reciprocating Mode T-LED curing light kit	0
	0
Instrument lever stop kit	0
6-way stainless steel syringe kit	•
6th instrument kit (curing light or camera handpiece)	0
ASSISTANT'S MODULE	
T-LED curing light kit	0
3-way stainless steel syringe kit	0
6-way stainless steel syringe kit	0
Removable and disinfectable suction tubes	•
Removable and autoclavable suction cannulae	•
Assistant's module with 4 housings and capacitive keypad mounted on self-balancing arm	•
Assistant's module with 5 housings and capacitive keypad mounted on self-balancing arm	0
Tray holder table on assistant's module	0
OPERATING LIGHT	
Venus LED MCT lamp	•
<u>UNIT BODY</u>	
Swivel and removable motor-driven cuspidor bowl with filling sensor	•
Water-to-cup heater	0
Instrument spray heater	0
Cannulae independent selection	•
Air/water supply quick couplings for external devices	0
Multimedia cables and PC connections	•
Tray holder table in place of bowl	0
PATIENT CHAIR	
SOFT-MOTION and Sliding feature	•
Pneumatic release headrest with 3-axis movement	•
Lateral slide foot control complete with joystick	•
Pressure-operated foot control with joystick and spray on/off kit	0
Foot control, Power Pedal type	0
Left armrest	0
	O.

● = supplied as standard O = optional - = not available